
REF	Désignation / Omschrijving Bezeichnung / Designation	Mesures / Maten / Maßnahmen / Measurements
RFL15087	PREMIUM LIMB HOLDERS	14 - 28 cm
RFL15600	COMFORT LIMB HOLDERS	14 - 28 cm
RFL15700	ECO COMFORT LIMB HOLDERS	14 - 28 cm
RFL15900	ECO LIMB HOLDERS	10 - 24 cm
RFW15100	LIMB HOLDERS	14 - 24 cm
RFW16000	COMFORT WRIST HOLDERS	10 - 24 cm
RFW16905	WRIST HOLDERS PEDIATRIC	0-2 ans jaar Jarhe years
RFW16910	WRIST HOLDERS PEDIATRIC	2 - 5 ans jaar Jarhe years

- Associer sangles poignets et chevilles avec ceinture abdominale et pelvienne.
- Combineer pols- enkels riemen met de buik- en bekkenriem.
- Kombinieren Sie Handgelenk- und Knöchelriemen mit dem Bauch- und Beckengurt.
- Associate wrists and ankles holders with the waist and pelvic belt.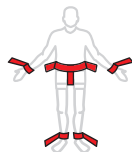

Lire la notice (IFU) / Lees de instructies (IFU) / Lesen sie die brochure (IFU) / Read instructions(IFU)

Regarder la Video / Bekijk Video / Schau Video / Watch Video

Utilisation au lit /gebruik in bed / im Bett verwenden/ using in bed

Ceinture poignet (paire) / Polsriem (paar) / Handhalterung (paar) / Wrist holder (pair)

Lit en position basse, barrière de lit en position haute, les patients doivent être observés d'une manière constante.
Bed in lage positie, bedhek in hoge positie, patiënten moeten constant worden geobserveerd.
Bett in niedriger Position, Bettgitter in hoher Position, Patienten sollten ständig beobachtet werden.
Bed in low position, bed rail in high position, patients should be observed constantly.

Ceinture cheville (paire) / Enkelriem (paar) / Fußhalterung (paar) / Ankle holder (pair)

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RENOL products are safety, immobilization, restraint products

Indications

1. RENOL products are used for medical purposes to secure an agitated patient in bed, on a stretcher, to avoid an accident, to allow the patient to follow his medical treatment.
2. RENOL products are only used under medical prescription.
3. The products must be used by trained staff following the RENOL's instructions for safe use and according to the establishment's safety rules.

Contraindications

4. Ensure that RENOL products do not come into contact with injured skin, do not obstruct breathing nor impede blood pressure.

Precautions of use

5. To avoid any accident and in order to ensure the person's comfort and safety, it is essential to read the instruction leaflets carefully and understand them before operating Renol products. All users follow the instructions and procedures, patients' safety depends on using belts and the locking system correctly.
6. Complications, accidents related to the restraint in case of misuse, during unsuitable, unsupervised use: bedsores, pneumonia or deep vein thrombosis, psychological stress (agitation, delirium) and negative impact on cognitive abilities, consequence fatal (following severe trauma or suffocation).
7. Ensure that a RENOL product is not damaged before use, all products must be clean and in good condition.
8. We recommend monitoring the immobilised patient frequently; restless and combative patients must be under constant observation.
9. Always use the correct size: too small or too large may compromise safety or comfort. Measure and select the appropriate RENOL product.
10. Remove all sharp objects from the patient (jewellery, watches, glasses, etc.).
11. Only use products RENOL on suitable support.
12. Connect RENOL products to a solid part of the chair. The straps must be secured; they cannot slide or change position; the fastening point of the strap on the frame equipment must not damage the product (sharp edge, etc.).

Maintenance and materials

13. Follow the washing instructions marked on the products.
14. Close Velcro fastenings before washing and drying to prevent them wearing.
15. Detergents: avoid using chlorine.
16. Temperature: maximum temperature 95°C.
17. Drying: delicate cycle at low temperature, about 5% shrinkage possible.
18. Material: 100% Terylene (polyester).
19. The raw materials are treated with fire retardant and anti bacterial.
20. 2 years warranty, this warranty does not cover damage resulting from normal wear and tear.

All our products are CE-approved - Class I medical devices.

Any serious incident occurring in connection with this device must be notified to the manufacturer and to the competent authority of the country where the serious incident took place.

For more information, contact your support specialist, your distributor or www.renolcare.com