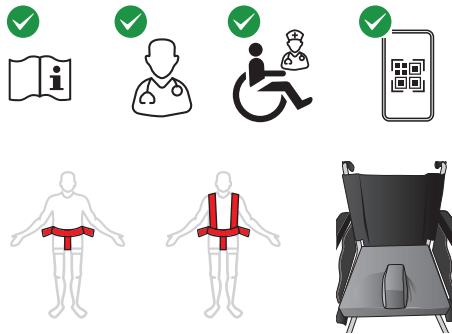
| REF      | Désignation / Omschrijving<br>Bezeichnung / Designation | Mesures / Maten /<br>Maßnahmen /<br>Measurements |
|----------|---------------------------------------------------------|--------------------------------------------------|
| RFC26000 | WAIST STRAP SOFT                                        |                                                  |
| RFC26100 | WAIST BELT SOFT                                         |                                                  |
| RFC26200 | WAIST AND PELVIC BELT SOFT                              |                                                  |
| RFC26300 | WAIST AND SHOULDER BELT SOFT                            |                                                  |
| RFC26400 | WAIST / SHOULDER AND PELVIC BELT SOFT                   |                                                  |
| RFS19050 | Extender nylon buckle 2.0                               | 150cm                                            |
| RFS20050 | Extender nylon buckle 4.0                               | 150cm                                            |

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RENOL products are safety, immobilization, restraint products

#### Indications

1. RENOL products are used for medical purposes to secure an agitated patient in the chair, to avoid an accident, to allow the patient to follow his medical treatment.
2. RENOL products are only used under medical prescription.
3. The products must be used by trained staff following the RENOL's instructions for safe use and according to the establishment's safety rules.

#### Contraindications

4. Ensure that RENOL products do not come into contact with injured skin, do not obstruct breathing nor impede blood pressure.

#### Precautions of use

5. To avoid any accident and in order to ensure the person's comfort and safety, it is essential to read the instruction leaflets carefully and understand them before operating Renol products. All users follow the instructions and procedures, patients' safety depends on using belts and the locking system correctly.
6. Complications, accidents related to the restraint in case of misuse, during unsuitable, unsupervised use: bedsores, pneumonia or deep vein thrombosis, psychological stress (agitation, delirium) and negative impact on cognitive abilities, consequence fatal (following severe trauma or suffocation).
7. Ensure that a RENOL product is not damaged before use, all products must be clean and in good condition.
8. We recommend monitoring the immobilised patient frequently; restless and combative patients must be under constant observation.
9. Always use the correct size: too small or too large may compromise safety or comfort. Measure the patient's waist size, select the appropriate RENOL product.
- 10.. Remove all sharp objects from the patient (jewellery, watches, glasses, etc.).
11. Only use products RENOL on suitable support. Never use RENOL products as a safety belt in vehicles.
12. When supporting or immobilising the patient in a chair, don't forget the patient's ability to turn over.
13. Connect RENOL products to a solid part of the chair. The straps must be secured; they cannot slide or change position; the fastening point of the strap on the frame equipment must not damage the product (sharp edge, etc.).
14. When immobilizing the patient in the chair with the RFC26100 and RFC26300 belt, we recommend the use of a RENOL Abduction cushion (RCV07300 – RCV07400) to prevent forward sliding and possible suffocation of the patient with his strap.

#### Maintenance and materials

15. Follow the washing instructions marked on the products.
16. Detergents: avoid using chlorine.
17. Temperature: maximum temperature 95°C.
18. Drying: delicate cycle at low temperature, about 5% shrinkage possible.
19. Material: 100% terylene (polyester).
20. The raw materials are treated with fire retardant and anti-bacterial.
21. 2 years warranty, this warranty does not cover damage resulting from normal wear and tear.

All our products are CE-approved – Class I medical devices.

Any serious incident occurring in connection with this device must be notified to the manufacturer and to the competent authority of the country where the serious incident took place.

For more information, contact your support specialist, your distributor or [www.renolcare.com](http://www.renolcare.com)