
REF	Désignation / Omschrijving Bezeichnung / Designation	Mesures / Maten / Maßnahmen / Measurements
RFK01111	MAGNET KEY (M)	
RFK01222	MAGNET BUTTON + PIN (M)	
RFK01201	MAGNET PIN	
RFK01202	MAGNET BUTTON	
RFK00100	STEEL KEY (RL)	
RFK00433	STEEL BUTTON + PIN (RL)	
RFK00300	STEEL PIN	
RFK00400	STEEL BUTTON	
REF + M =	BELT + MAGNET LOCKS	
REF + RL =	BELT + STEEL LOCKS	

REF + M =


REF= RFK01111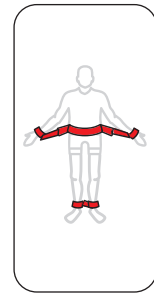

REF= RFK01222

REF + RL =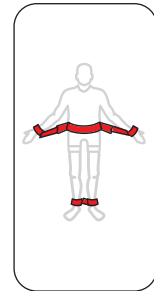

REF= RFK00100

REF= RFK00433

Lire la notice (IFU) / Lezen IFU / Lesen IFU / Read IFU

Regarder la Video / Bekijk Video / Shau Video / Watch Video

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Indications

1. RENOL products are used for a medical purpose to ensure the safety and security of the patient.
2. RENOL products are only used under medical prescription.
3. The products must be used by trained staff following the RENOL's instructions for safe use and according to the establishment's safety rules.

Precautions of use

4. To avoid any accident and in order to ensure the person's comfort and safety, it is essential to read the instruction leaflets carefully and understand them before operating belts. All users follow the instructions and procedures, patients' safety depends on using belts correctly.
5. Ensure that a RENOL product is not damaged before use, all products must be clean and in good condition.
6. We recommend monitoring the immobilised patient frequently; restless and combative patients must be under constant observation.
7. Ensure that RENOL products do not come into contact with injured skin, do not obstruct breathing nor impede blood pressure.
8. Always use the correct size: too small or too large may compromise safety or comfort. Measure the patient's waist size, wrist and ankle, select the appropriate RENOL product.
9. Remove all sharp objects from the patient (jewellery, watches, glasses, etc.).
10. When supporting or immobilising the patient, don't forget the patient's ability to turn over.
11. Ensure that the RENOL key, part number: RFK00100 (RL) and RFK01111 (M) is always available. Use only RENOL locks.
12. In an emergency, if the key is not available or problem with lock system, cut the strap.
13. Before use, check operation of the magnet (M) lock system, check correct operation of the RFK01202 Button with the key RFK01111 and pin RFK01201. We recommend replacing the Button RFK01202 after 2 years. The lifetime may be shorter or longer, depending on use.
14. Before use, check operation of the mechanical lock system (RL), check correct operation of the RFK00400 Button with the key RFK00100 and long pin RFK00300. We recommend replacing the Button RFK00400 every 3 years. The lifetime may be shorter or longer, depending on use.
15. The magnetic key (M) must be placed 10 cm minimum from a pacemaker or any other medical system that could be disturbed by a magnet.
16. The magnetic lock system could be activated with a magnet other than that delivered by RENOL.

Maintenance and materials

20. Detergents: avoid using chlorine.
21. Disinfection by surface cleaning
22. Remove all locking systems before washing and drying.
23. Never immerse the magnetic (M) and mechanical closures (RL) in a liquid.
24. 2 years warranty, this warranty does not cover damage resulting from normal wear and tear.

All our products are CE-approved - Class I medical devices.

Any serious incident occurring in connection with this device must be notified to the manufacturer and to the competent authority of the country where the serious incident took place.

For more information, contact your support specialist, your distributor or www.renolcare.com.