REF

Désignation / Omschrijving
Bezeichnung / Designation

RCP01235	HAND PROTECTION RIGID (NET)
RCP01238	HAND PROJECTION RIGID
RCP01236	HAND PROTECTION RIGID WITH FINGER SEPARATORS

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Wrap the velcro of the glove around the wrist

Les patients doivent être observés d'une manière constante /
Patiënten moeten constant worden geobserveerd /
Sollten ständig beobachtet werden /
Patients should be observed constantly

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Made from cotton, the surface of this glove, on the palm side, remains rigid thanks to a plastic material, the outside is covered with a net which allows the skin to breathe easily.

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Made from cotton, the surface of this glove, on the palm side, remains rigid thanks to a plastic material, the outside is covered with a thin and strong cotton which allows the skin to breathe easily.

RCP01236 HAND PROTECTION RIGID WITH FINGER SEPARATORS

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Made from cotton, the surface of this glove, on the palm side, remains rigid thanks to a plastic material, the outside is covered with a net and a thin and strong cotton which allows the skin to breathe easily, finger separators reduce dexterity.

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Indications

These gloves are intended to prevent the patient from injuring themselves, voluntarily or otherwise, or from interfering with the care provided by pulling at dressings and probes.

Option

If the patient still wants to remove his gloves using his mouth, there are two rings in which a RENOL lock can be attached, possibility of immobilizing the wrist with a strap thanks to the ring on the protective glove, Doctor's prescription needed.

Materials

Cotton and Polyester

Maintenance

Machine washable. All care instructions are specified on the products.

Any serious incident occurring in connection with this device must be notified to the manufacturer and to the competent authority of the country where the serious incident took place.

For more information, contact your support specialist, your distributor or www.renolcare.com

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