
REF	Nome / Nombre Designação / Designation	Misure / Medidas / Medidas / Measurements
RFL15087	PREMIUM LIMB HOLDERS	14 - 28 cm
RFL15600	COMFORT LIMB HOLDERS	14 - 28 cm
RFL15700	ECO COMFORT LIMB HOLDERS	14 - 28 cm
RFL15900	ECO LIMB HOLDERS	10 - 24 cm
RFW15100	LIMB HOLDERS	14 - 24 cm
RFW16000	COMFORT WRIST HOLDERS	10 - 24 cm
RFW16905	WRIST HOLDERS PEDIATRIC	0-2 anni / años / anos years
RFW16910	WRIST HOLDERS PEDIATRIC	2 - 5 anni / años / anos years

- Associare sempre cintura addominale e cintura scapolare a una cintura pelvica / Fissare entrambi i polsi ed entrambe le caviglie con la cintura addominale e pelvica.
- Combine el cinturón abdominal y el cinturón de hombro siempre con un cinturón pélvico / Asegure ambas muñecas y ambos tobillos con el cinturón abdominal y pélvico.
- Associar o cinto abdominal e o cinto para costas sempre com um cinto pélvico / Apertar os dois pulsos e os dois tornozelos com o cinto abdominal e o cinto pélvico.
- Associate waist belt and shoulder belt always with a pelvic belt / Secure both wrists and both ankles with the waist and pelvic belt.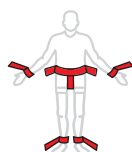

Leggere le istruzioni (IFU) / Leer el prospecto (IFU) / Ler as instruções (IFU) / Read instructions (IFU)

Guarda il video / Ver el vídeo / Ver o vídeo / Watch Video

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RENOL products are safety, immobilization, restraint products

Indications

1. RENOL products are used for medical purposes to secure an agitated patient in bed, on a stretcher, to avoid an accident, to allow the patient to follow his medical treatment.
2. RENOL products are only used under medical prescription.
3. The products must be used by trained staff following the RENOL's instructions for safe use and according to the establishment's safety rules.

Contraindications

4. Ensure that RENOL products do not come into contact with injured skin, do not obstruct breathing nor impede blood pressure.

Precautions of use

5. To avoid any accident and in order to ensure the person's comfort and safety, it is essential to read the instruction leaflets carefully and understand them before operating Renol products. All users follow the instructions and procedures, patients' safety depends on using belts and the locking system correctly.
6. Complications, accidents related to the restraint in case of misuse, during unsuitable, unsupervised use: bedsores, pneumonia or deep vein thrombosis, psychological stress (agitation, delirium) and negative impact on cognitive abilities, consequence fatal (following severe trauma or suffocation).
7. Ensure that a RENOL product is not damaged before use, all products must be clean and in good condition.
8. We recommend monitoring the immobilised patient frequently; restless and combative patients must be under constant observation.
9. Always use the correct size: too small or too large may compromise safety or comfort. Measure and select the appropriate RENOL product.
10. Remove all sharp objects from the patient (jewellery, watches, glasses, etc.).
11. Only use products RENOL on suitable support.
12. Connect RENOL products to a solid part of the chair. The straps must be secured; they cannot slide or change position; the fastening point of the strap on the frame equipment must not damage the product (sharp edge, etc.).

Maintenance and materials

13. Follow the washing instructions marked on the products.
14. Close Velcro fastenings before washing and drying to prevent them wearing.
15. Detergents: avoid using chlorine.
16. Temperature: maximum temperature 95°C.
17. Drying: delicate cycle at low temperature, about 5% shrinkage possible.
18. Material: 100% Terylene (polyester).
19. The raw materials are treated with fire retardant and anti bacterial.
20. 2 years warranty, this warranty does not cover damage resulting from normal wear and tear.

All our products are CE-approved - Class I medical devices.

Any serious incident occurring in connection with this device must be notified to the manufacturer and to the competent authority of the country where the serious incident took place.

For more information, contact your support specialist, your distributor or www.renolcare.com